


**ALPHA/BETA/GAMMA INFANT HAND***The Greek Series  
Right and Left Models, 3 Colors*

Patent # 5,085,665

**!** TRS recommends sterilizing these products before use.

Pacifier uses LATEX nipples. Do not use pacifiers if infant or family members are latex sensitive.

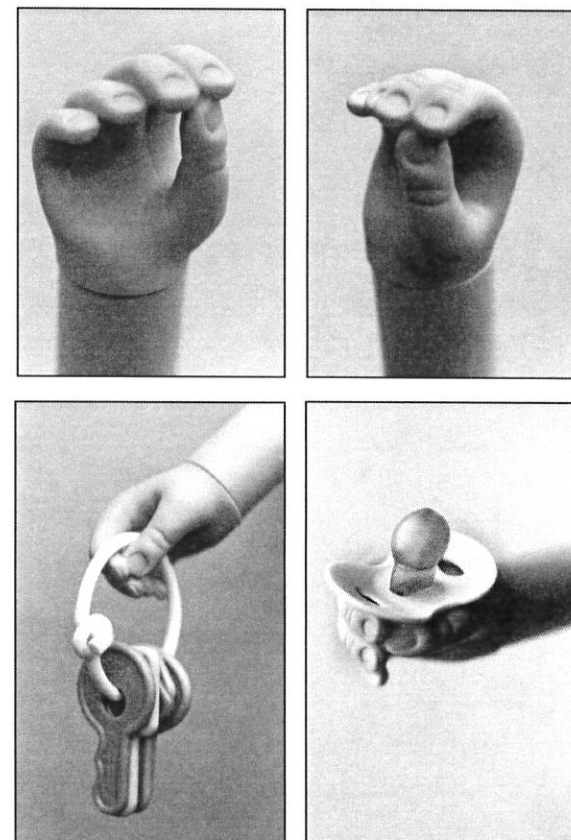

**PROSTHETICS RESEARCH ■ DESIGN ■ MANUFACTURING ■ CONSULTING**

3090 STERLING CIRCLE, STUDIO A, BOULDER, CO, USA 80301-2338  
 PHONE: 303.444.4720 TOLL FREE: 800.279.1865  
 FAX: 303.444.5372  
 ON THE WEB: [www.oandp.com/trs](http://www.oandp.com/trs)

---

## Limited Warranty

---

Products manufactured by TRS, Inc. are covered by a written Limited Warranty for a period of one year from the date of purchase.

This Limited Warranty covers any defects in workmanship or materials by TRS, Inc. for the period stated, provided the product is used in a reasonable manner. The Limited Warranty is not applicable to normal wear and not applicable to spring or spring-like components. The Limited Warranty applies only to the end user/purchaser.

Any implied warranties, including merchantability, are also limited to one year from the date of the original TRS Invoice and any action must be commenced within that period. This Limited Warranty is void if the product is misused or abused, if unauthorized alterations are made, or if maintenance is neglected.

TRS, Inc. shall not be liable for any incidental or consequential damages, unless otherwise required by law. Some states do not allow the exclusion or limitation of incidental or consequential damages or the limitation on the period the implied warranty lasts so that the above limitation or exclusion may or may not apply to you. This warranty gives you specific legal rights, and you may also have other rights, which vary from state to state.

Product Serial #: \_\_\_\_\_

## Installation:

---

The Greek Series Infant Hands are supplied with two options for mounting. The hand has a 1/2 inch diameter threaded receptacle molded into its base. The receptacle can be used alone if a standard wrist unit is not desired. Terminate the prosthesis in a 1/2 inch diameter threaded stud and The Greek Series Infant Hands can be screwed down upon it. Additionally TRS provides a 1/2 inch diameter threaded stud. The stud fits into the receptacle to provide a standard USA style wrist mounting.

## Training/Education Information:

---

The Greek Series Infant Hands are simple but highly functional prosthetic devices. They are flexible, durable and easy to use. The hand is designed for crawling and balance, bi-manual holding and uni-lateral holding options. It provides a safe, flexible interface at the end of the prosthesis. Safe, age appropriate objects can be inserted into the opening to help the child learn that a prosthesis can be used for holding. TRS provides a pacifier (ALPHA only) which will "force fit" into the opening. The pacifier (ALPHA only) and opening are sized to provide a secure fit so that the pacifier will not easily fall from the hand's grasp.



**Never place objects into the hand that could cause injury.**

**Never insert hard or sharp objects into the hand. The child may fall upon the hand and prosthesis or bring the hand to its face or near its eyes.**

**Always use caution and always supervise object handling.**

No formal training regime exists. Consult with a Registered Occupational Therapist (OTR) or Registered Physical Therapist (RPT) with specific experience in upper extremity prosthetics for specialized help and training.

## Care and Cleaning:

---



**TRS recommends sterilizing the ALPHA Infant Hand and pacifier before initial use. Sterilize by placing in boiling water for one to two minutes.**

**Note: The pacifier nipple is constructed of latex! If your child or family members are latex sensitive do not use the pacifier.**

The ALPHA, BETA, GAMMA Infant Hands are constructed of a durable polyurethane material. Polyurethane is tough, abrasion and chemical resistant. It resists dirt and stains but permanent dyes will discolor the material.

The ALPHA, BETA, GAMMA Infant Hands are designed expressly for infant use. The child will most likely place the hand in its mouth. Always keep the hand and pacifier clean. Clean with mild soaps or detergents and always rinse the ALPHA, BETA, GAMMA Infant hand and pacifier thoroughly to remove all cleaning product residues. Re-sterilize the hand and pacifier as necessary.